
#### Agrish Bed Rail with Light

## **Instruction Manual**

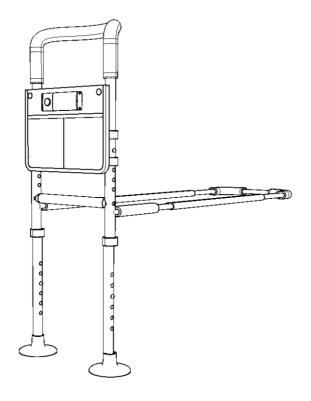



Email:

Note:

# Which Installation is Correct for You?

Bed rail should be kept in placement Avoid damage.

Please make sure that the bed rails fit closely to the mattress.






**⚠** BE CAREFUL!

### **Motion Light Tips:**

- \*The sticky magnets must be attached to the middle of the light.
- 2. \*Paste the magnetic sticky on the middle to the storage pocket or the installation position.
- 3. \*Bed rail should be kept in placement Avoid damage.
- ▲ Please make sure that the bed rails fit closely to the mattress.

Agrish.us@outlook.com

# PART LIST

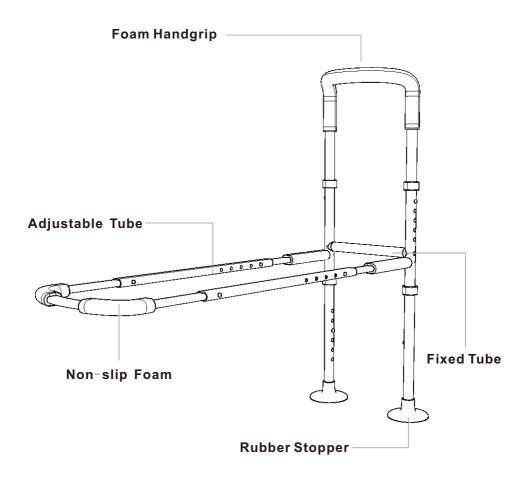


Part Number	Diagram	Description	Qty
#A		Foam Handgrip	1
#B	0 0 0 0	<b>L-Shape Tube</b> (2 of the same)	2
#C		Fixed Tube	1
#D		Flat Tube (2 of the same)	2
#E		U-Shape Frame	1
#F	0000000	Support Leg	2

MISSING PARTS, OR REPLACEMENT **PLEASE CONTACT US** TO SOLVE WE'D LOVE TO HELP YOU ANYTIME.


Agrish.us@outlook.com

# **12** HOW TO INSTALL?


1. First, we will assemble according to the corresponding area of the letter. As shown below.

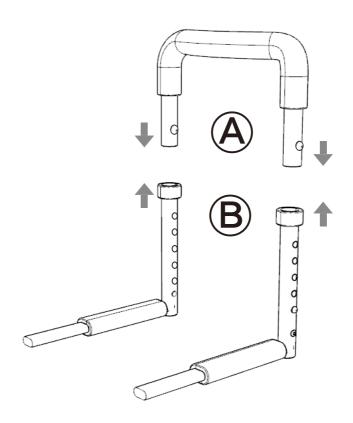


2. Insert 2 black rubber and 2 screws into the crossbar respectively, and tighter them with a hexagon spanner.

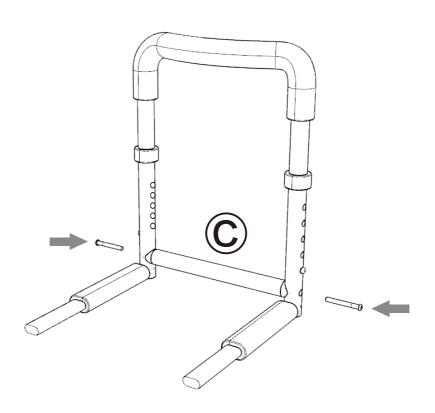


3. Insert 2 legs into the L-Shaped Tube, making sure that the adjustable holes remain inthe same direction as shown in the picture.

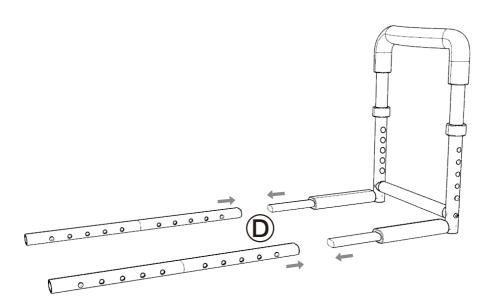


4. As shown in the figure, insert 2 Flat Tubes into the U-Shaped Frame.

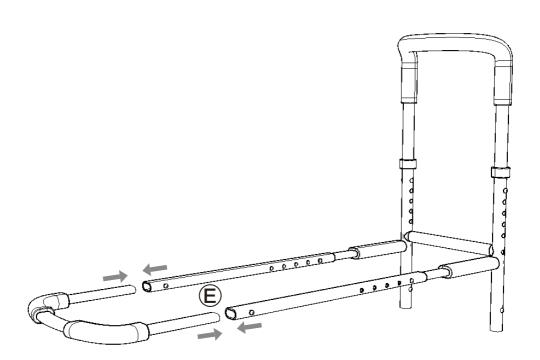

5. Insert support leg (#F) into the bottom of frame (#B) by compressing the silver button and locking it into place.

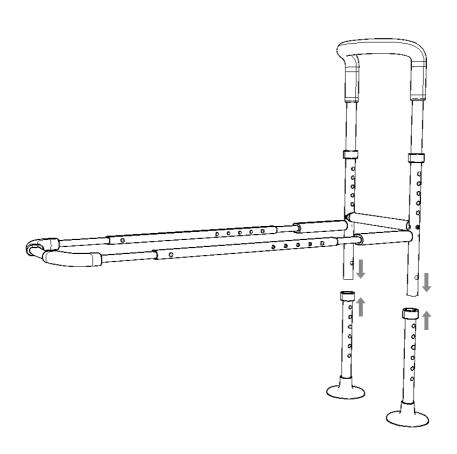

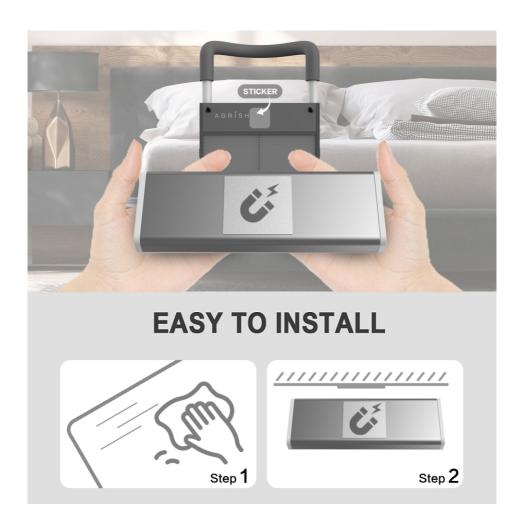

#### NOTE:

- 1. The magnets sticky must be attached to the middle of the light.
- 2. Please put the magnetic sticky on the middle and then paste it to the storagebag or the installation position.

## TIPS

Q: How do you prevent water in the tollet from splashing up to your butt when you make your regular deposit?

A: Place a single square of toilet paper in the center of the bowel, right where you expect your 'deposit' to fall. Then let it drop! There will be no splash.

#### Cleaning/Maintenance

Wipe the entire chair with a neutral detergent, then wipe with clean water, and then dry it in a ventilated place.

#### Prevention and warning

- 1. The product should not be left outside on rainy days.
- 2. Before sitting down, make sure all these buckles are locked in the correct position.
- 3. Do not hang heavy objects on either side of the armrest, otherwise the commode chair will become unstable.

#### 180-DAY Warranty

AGRISH is warranted to be free of defects in materials and workmanship. This device was built to exacting standards and carefully inspected prior to shipment. This Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about this toilet device or this warranty, please contact us.

# MORE AGRISH PRODUCTS FOR YOUR NEEDS



Bed Rails with Legs

Bedside Assist Bars





Toilet Safety Rails



Rised Toilet Seat

3-IN-1 Bedside Commode





Walker Ski Glides



#### Welcome to Agrish Store:

https://www.amazon.com/stores/AgrishHealth/page/ 26E5C6BF-9D4B-4CA4-8C89-6E478C089448

### AGRISH, A great wish you go well.:)

#### PERFECT / LESS PERFECT?

Let us know with a customer review.

Agrish is committed to delivering customer-driven products that live up to your high standards. We are really appreciate if you can share your honest experience for our product on Amazon by: ☐ https://www.amazon.com/review/review-your-purchases



180-DAY Manufacturer's Warranty in Canada & US

**Customer Service & Support** 

Agrish.us@outlook.com