

AGRISH Rised Toilet Seat Instruction Manual



Email:

Agrish.us@outlook.com


Note:

Please read the instructions carefully before use

WARNINGS








- DO NOT install commode without first reading and understanding this assembly and instruction sheet. If you can not understand these instructions, contact us directly before attempting to install. Otherwise injury or damage may occur.
- DO NOT exceed the maximum user weight capacity of 350 lbs.
- Inspect all parts for shipping damage before assembly. If there is any shipping damage, DO NOT USE. Contact us/carrier for further instructions.
- Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
- Check the rubber tips on the leg extensions for rips, wear or if they are missing, please contact us.
- All four leg extensions with rubber tips must touch the floor simultaneously at all times.
- Users with limited physical strength should be supervised or assisted while using commode.

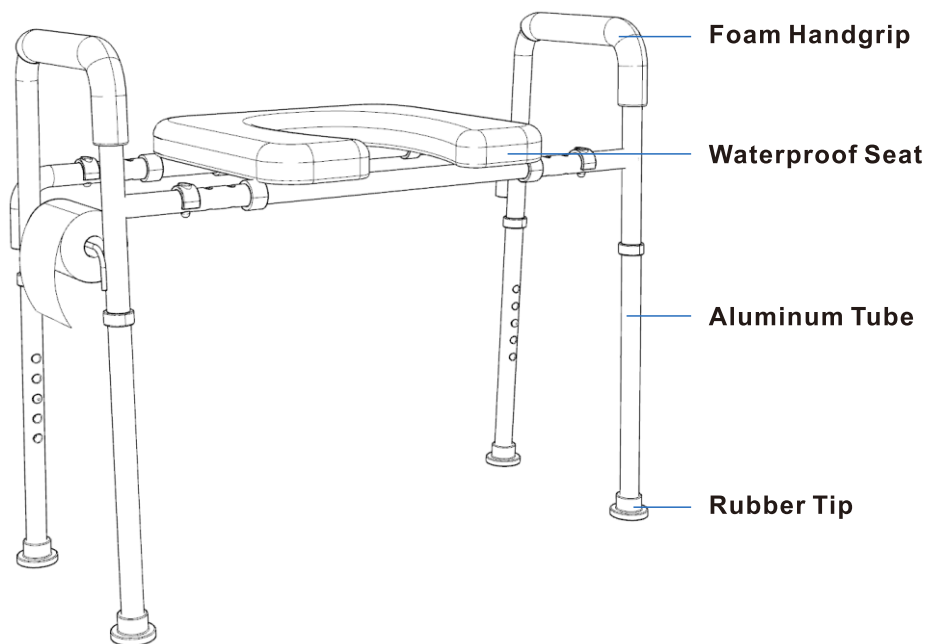01 PART LIST



Easy Assemble VIDEO on YouTube:
'How to Setup the AGRISH Raised Toilet Seat'

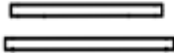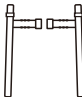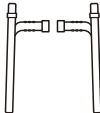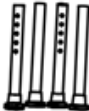
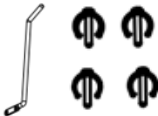


Part Number	Diagram	Description	Qty
#A		Armrest	2
#B		Padded Seat	1
#C		Support Tube	2
#D		Front Frame	2
#E		Back Frame	2
#F		Leg Tube	4
#G		Hook & C Clips	1

MISSING PARTS, OR REPLACEMENT
PLEASE CONTACT US TO SOLVE
WE'D LOVE TO HELP YOU ANYTIME.


02 HOW TO INSTALL?

1. First, we will assemble according to the corresponding area of the letter. As shown below.

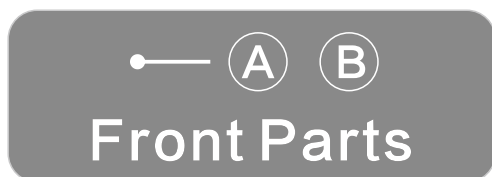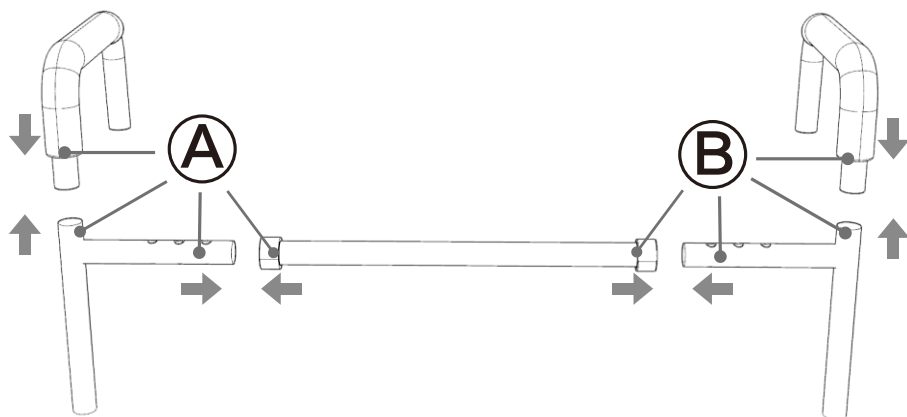

NOTE: There are 3-width settings to fit your body.


Setup Back Side

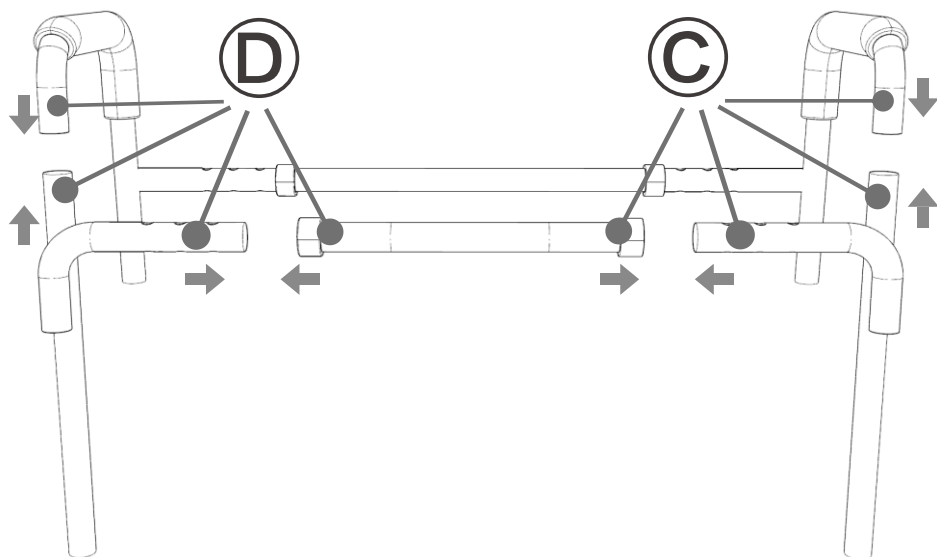


● — (C) (D)

Back Parts


2. According to your body and the room of the toilet, choose the right and left holes. Then insert the fixing pins (C-clips #G) into the four places as picture showed and fix them.

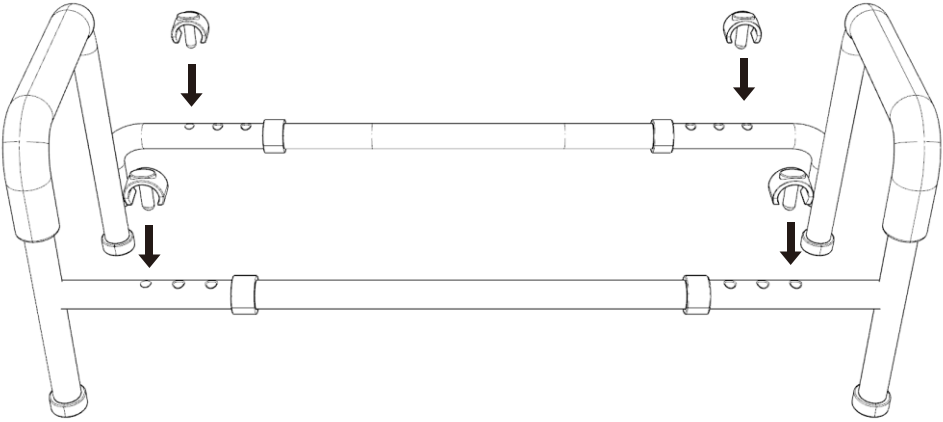


3. Install the four legs. And adjust buttons on the same level to keep balance.

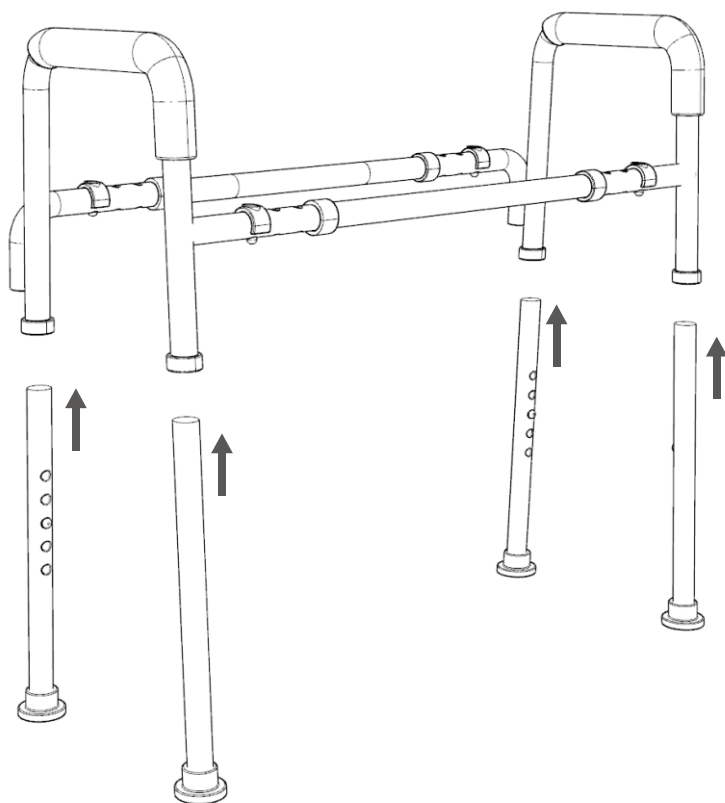

4. And then install the seat plate according to the below picture.

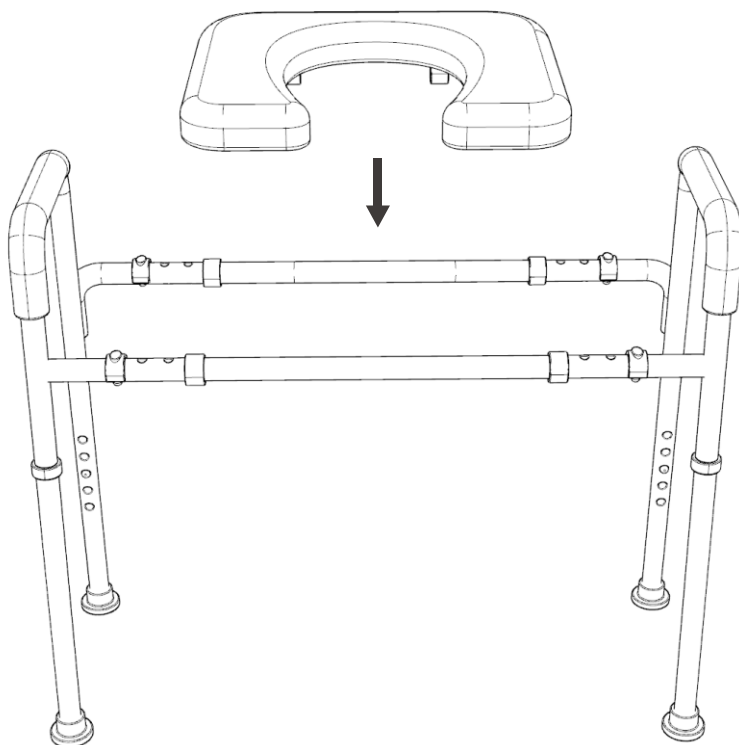

5. Finally, use a hex wrench to install the paper hook into the side of tube. (Which side to install according to your preference.)

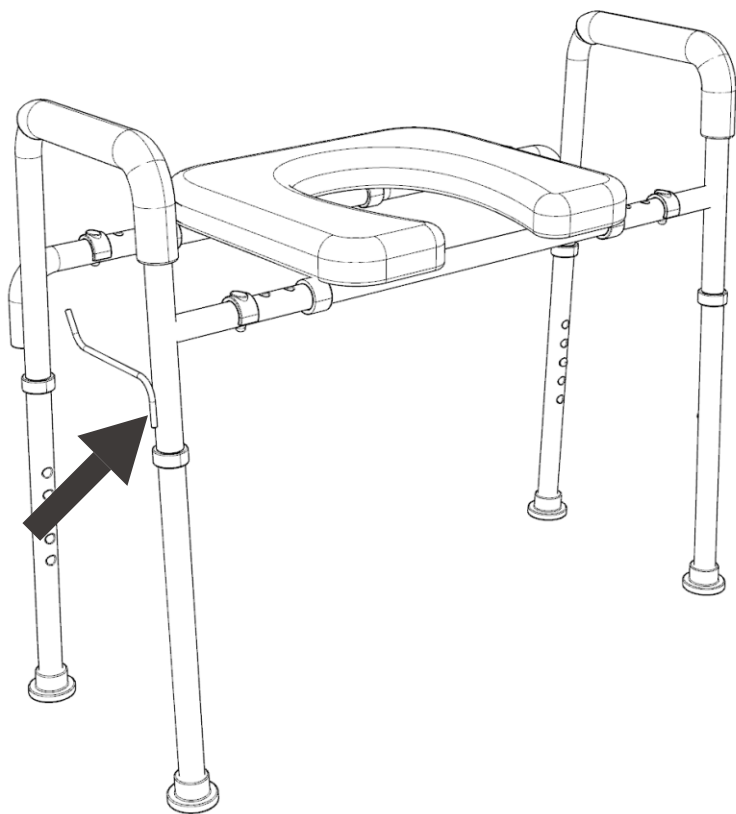

TIPS

Q: How do you **prevent water in the toilet from splashing** up to your butt when you make your regular deposit?

A: Place a single square of toilet paper in the center of the bowl, right where you expect your 'deposit' to fall. Then let it drop! There will be no splash.

Cleaning/Maintenance

Wipe the entire chair with a neutral detergent, then wipe with clean water, and then dry it in a ventilated place.

Prevention and warning

1. The product should not be left outside on rainy days.
2. Before sitting down, make sure all these buckles are locked in the correct position.
3. Do not hang heavy objects on either side of the armrest, otherwise the commode chair will become unstable.

180–DAY Warranty

AGRISH is warranted to be free of defects in materials and workmanship. This device was built to exacting standards and carefully inspected prior to shipment. This Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

The warranty does not extend to non–durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about this toilet device or this warranty, please contact us.

MORE AGRISH PRODUCTS FOR YOUR NEEDS



**Bed Rails
with Legs**

**Bedside
Assist Bars**




**Toilet
Safety Rails**



Rised
Toilet Seat

3-IN-1
Bedside
Commode

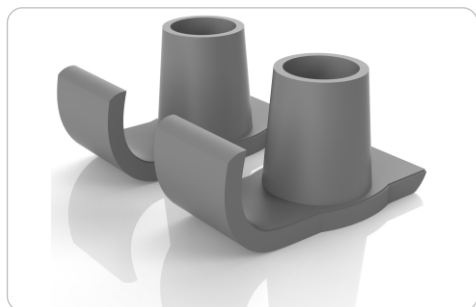

Walker
Ski Glides



Welcome to Agrish Store:

<https://www.amazon.com/stores/AgrishHealth/page/26E5C6BF-9D4B-4CA4-8C89-6E478C089448>

AGRISH, A great wish you go well. :)

Thank you for your Agrish Products purchase!

We hope you love it as much as we loved making it.

Please leave us a review and let us know how we did.



Scan QR code to share your shipping experience.

**180-DAY
Manufacturer's
Warranty in Canada & US**

Customer Service & Support
Agrish.us@outlook.com



SAVE THE PLANET!



Returns generate more than 15 million tons of CO₂, actually.



If pieces are missing or have arrived damaged, before return, contact us, please.



Agrish.us@outlook.com